



*Designed for Form,  
Function & Flow*



888-760-8159 • www.cygnusinc.net • 316 Suite One E. Mitchell St, Petoskey MI, 49770

## Push Button Lock

### Operation


- Buttons generally run 1-5 from left to right.
- Enter the appropriate number combination, turn the knob clockwise and hold it while you open the door.
- If a code is entered in error clear it by turning the knob counter-clockwise.

### Standard Combinations

- ChartRak: Press [2] and release. Press [4] and release.
- Medication Box: Press [2] and [4] simultaneously then release them. Press [3] and release.

### Change Codes

- Only turn the knob when you are instructed to do so.
- Clear the existing programmed code by opening the cabinet, and then repeat the same push button sequence as in previous step with the door open, but **DO NOT turn knob!**
- Press the metal tab on the left side of the lock mechanism inward toward to depress the plunger.
- Turn the knob on the face of the lock counter-clockwise and then release it. The code is now cleared.

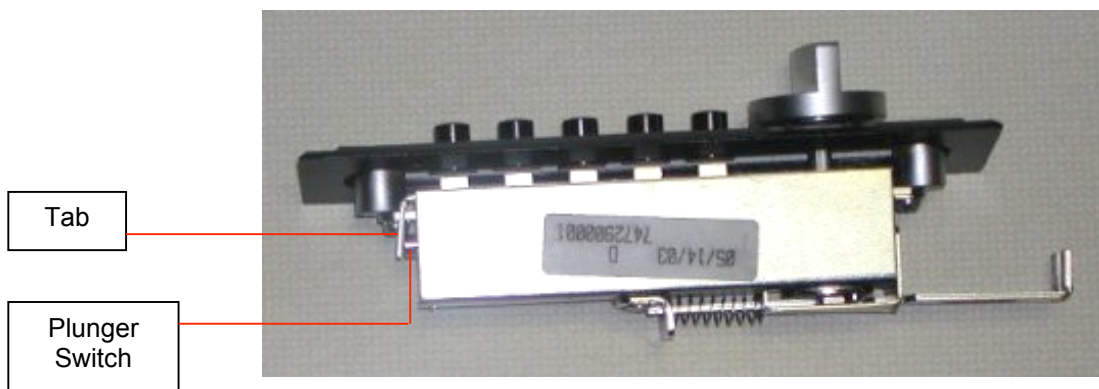

- To enter a new code enter your combination and turn the knob clockwise. The plunger switch will extend and you will hear a click. The new code is now set. Record the new combination.